



Battery Operated Nasal Aspirator

USER MANUAL

INTENDED USE : Quickly and gently removes excess nasal mucus.

REF **SNOTTY TOO**

Warnings:

Before using this device, please do read all instructions in this manual. Please keep this manual as it contains important information.

1. This device is not a toy; do not allow children to play with it.
2. This device is to clean stuffy noses; do not use it in baby's mouth or ears.
3. Always clean and dry thoroughly before each use and after each use.
4. This is not a waterproof device, please clean it referring to the section of "Clean Instructions".
5. Do not insert the silicone tip into the nostril too deep.
6. Do not use it on the baby for a long time.
A few seconds will be enough.
7. Do not use this device if there is any wound, or symptom of inflammation or bleeding at the nose.
8. In the case of infection, fever, nasal irritation or hemorrhage, please stop using this device and consult your physician.
9. This device is restricted for personal use.
10. The silicone tip is the applied part.
11. No modification of this device is allowed.
12. Duty cycle : 30 seconds ON / 2 minutes OFF.
13. Replace batteries when the suction power becomes weak during use.

Battery Warnings:

Use alkaline batteries only

1. Do not mix used and new batteries.
2. Do not mix alkaline, standard (carbon zinc), or rechargeable (nickel-cadmium) batteries.
3. Only batteries of the same or equivalent type as recommended should be used.
4. Batteries should be inserted with correct polarity.
5. Exhausted batteries should be removed from the device.
6. Be sure that all contact surfaces are clean before installing batteries.
7. Risk of explosion if battery is replaced by an incorrect type.
8. Remove batteries when stored for long periods of non-use.

- 1 -


Contents:

- 1 Aspirator
- 2 Silicone Tips
- 1 Instruction Manual
- 1 Cleaning Tube
- 2 Spare Silicone Rings

Not included:

- 2 x 1.5V AA Alkaline Batteries (Required for operation)

Parts Description

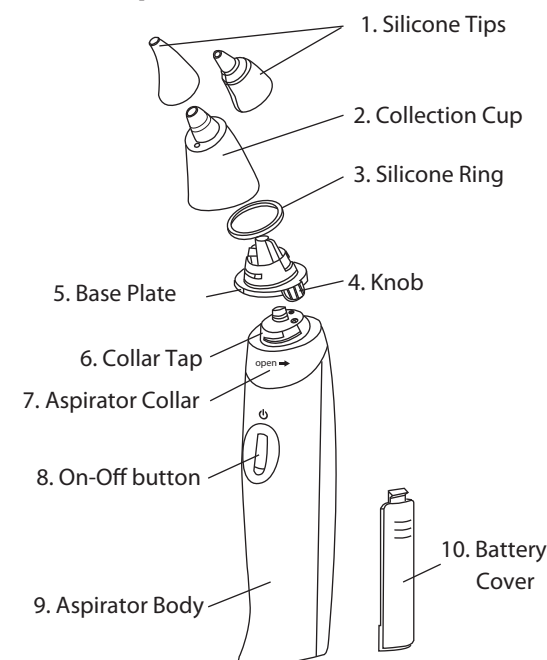

- 2 -



is proud to offer Australian parents WORK FROM HOME opportunities with the goal of improving their financial situation while promoting children's health and welfare.

The concept is simple:

1. You purchase aspirators from us
2. You sell them; wholesale, retail, web sales or by hosting *SNOTTY* parties
3. ALL takings from sales are yours, earning as much as 300% on your investment

workfromhome@snotty.com.au

Applied Standards:

This device conforms to the provisions of the EC directive MDD (93/42/EEC). The following standards apply to design and/or manufacturing of the device:

- IEC / EN 60601-1 Medical electrical equipment -- Part 1: General requirements for basic safety and essential performance
- IEC / EN 60601-1-2 Medical electrical equipment -- Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic compatibility - Requirements and tests
- ENISO 10993-5 Biological evaluation of medical devices -- Part 5: Tests for in vitro cytotoxicity
- ENISO 10993-10 Biological evaluation of medical devices -- Part 10: Tests for irritation and delayed-typed hypersensitivity

SNOTTY TOO - 20131101 - V1

Initial Use:

1. Remove the battery cover.
 2. Insert 2 x 1.5V AA alkaline batteries.
 3. Replace the battery cover.
4. Select a suitable silicone tip based on the size of your child's nostrils. (Inserting tip as little as possible into nostril, while sealing as much as possible of the nostril.)
5. Be sure to place the silicone tip securely on the mucus cap.

6. Spray saline into child's nostril and wait for a minute or two., then gently aspirate your child.

- 3 -

7. Pressing the on-off button for a few seconds to aspirate your child.

Cleaning Instructions :

1. Remove the silicone tip
2. Use the finger and thumb to hold the 2 knobs of the base plate and unscrew the base plate from the main unit by turning it counterclockwise as "open →" on the collar and then pull up.
3. Hold the collection cup upside down. Use the finger and thumb to hold the 2 knobs of base plate and turn it clockwise as indicated by "open →" on the the base plate.
4. Remove the silicone ring from the base plate.

- 4 -


5. Wash all 4 parts in warm soapy water and then clean them with steam or boiling water.

6. Assemble all the parts as shown below:

7. Be sure all the parts fit tightly together and line up the indication marks.

If you find mucus over the collar tap, please follow below instructions.

1. Attach the cleaning tube by putting it into the suction hole.

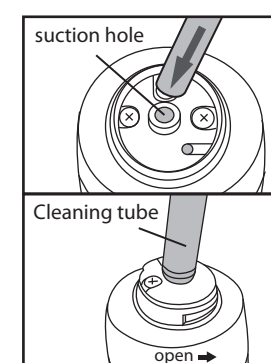

- 5 -

2. Place the tip of the cleaning tube in soapy water and press the on-off button for about 10-20 seconds.

3. Lift the cleaning tube from the water and press the button again to pump out water from inside aspirator. Remove the cleaning tube, keeping it for future use.

Operation ,Storage & Transportation Environment Information:

Operation environment	Temperature: 10°C~40°C Humidity: ≤95%
Storage & Transportation environment	Temperature: -10°C~50°C Humidity: ≤95%

MANUFACTURED FOR:
Perpetus Pty Ltd Australia
ABN 49 142 811 069
T/A SNOTTY
PO Box 111
Labrador QLD 4215

EC REP AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
Obelis s.a.
Boulevard General Wahis 53
1030 Brussels, BELGIUM



LOT BATCH CODE
SEE THE LOT LABEL AT THE BOTTOM OF THE DEVICE

CONSULT INSTRUCTIONS FOR USE

Type BF applied part

- 6 -